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106. 12. 13

Perfect^{TCS II}

The state-of-the-art electrosurgical system for precise and efficient tissue contouring

Perfect^{TCS II}
Tissue Contouring System

Perfect^{TCS II} instructional aids included with each unit:
▶ Introductory CD ▶ Step-by-step Owner's Guide

Ordering Information

Perfect ^{TCS II} 115V		
Cat. No.	Qty.	Electrode Sheaths
Included Components		
Perfect TCS II Unit 115V	1	Complete Set S6011 thru S6018
Cord Assembly S6008CE	1	Included Set S6011, S6012, S6015
Electrode Set S7010A	3	Coag Ball
Dispersive Electrode S213CE	1	S6012A 45° Straight Knife
Handpiece Holder S7001	1	S6013A 45° Tapered Knife
Cat. No. S8115T		
Perfect TCS II Unit 115V	1	S6014A Straight Round Loop
Cord Assembly S6008CE	1	S6015A 45° Long Loop
Electrode Set S6010A	8	S6016A Straight Long Loop
Dispersive Electrode S213	1	S6017A 45° Diamond Loop
Handpiece Holder S7001	1	S6018A 90° AP 1 1/2

Coltène / Whaledent Inc.

235 Ascot Parkway
Cuyahoga Falls, OH 44223 / USA
Tel. 1 800 221 3046
Tel. USA & Canada 1 800 221 3046
Fax +1 330 916 8800
info@coltene-whaledent.com

Coltène / Whaledent AG

Feldwegstrasse 20
9450 Albstadt / Switzerland
Tel. +41 (0) 71 757 53 00
Fax +41 (0) 71 757 53 01
office@coltene-whaledent.ch

Authorized EU Representative:

Coltène / Whaledent GmbH + Co. KG
Raiffeisenstrasse 30
89129 Langenau / Germany
Tel. +49 (0) 7345 805 0
Fax +49 (0) 7345 805 201
productinfo@coltene-whaledent.de

Coltène / Whaledent S.A.r.l.

19 Boulevard Alexandre Oyon
72058 Le Mans Cedex 2/ France
Tel. +33 (0) 43 39 30 30
Fax +33 (0) 43 39 30 40
cwo@coltene-whaledent.fr

Coltène / Whaledent

C/ Buenavista, 3-1 B
28220 Majadahonda
Madrid, Spain
Tel. +34 91 638 64 92
Fax +34 91 638 62 79
coltene-whaledent@telefonos.es

Coltène / Whaledent Ltd.

The President Suite-A, Kendal House
Burgess Hill, Victoria Way
Wrest Sussex, RH15 9NFK
Tel. +44 (0)1444 235486
Fax +44 (0)1444 870640
hw@coltene-whaledent.co.uk

Coltène / Whaledent website:

<http://www.coltene-whaledent.com>

For Export Only
Made in USA

CE 0344

coltene whaledent

If you think this is the cutting edge
of tissue management,

read on...

coltene whaledent

How to make everyday procedures simpler and clinically superior

Perfect® TCS II, a high frequency electro-surgery system, helps manage soft tissue with greater efficiency, greater precision and less bleeding than the common dental scalpel

- ▶ Removes unwanted tissue with ease
- ▶ Reduces chair time by simplifying operative procedures and maintaining hemostasis
- ▶ Provides unequalled cutting precision for superior clinical and aesthetic results
- ▶ Allows uneventful healing

Other clinical applications:

- ▶ Widening gingival sulcus
- ▶ Crown lengthening
- ▶ Exposing impacted teeth
- ▶ Incising abscesses
- ▶ Removing hyperplastic gingiva
- ▶ Frenectomy

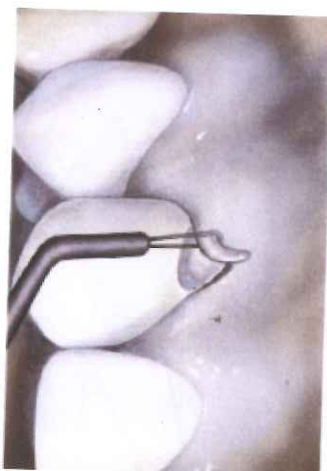
Using the Straight Knife electrode to stop bleeding in gingival sulcus

Control bleeding with a touch of the electrode

- ▶ Perfect® TCS II quickly stops bleeding caused by tooth preparation or cord placement with just a few dabs of the electrode
- ▶ The procedure continues without delay — no more waiting or extra appointments

Coagulation in gingival trough


Using the Long Loop electrode to remove overhanging gingiva

Gain access to caries quickly and efficiently

- ▶ With Perfect® TCS II, you can quickly remove gingival tissue occluding carious lesions
- ▶ Restorations — even with composites — may begin immediately because the operative field is blood-free

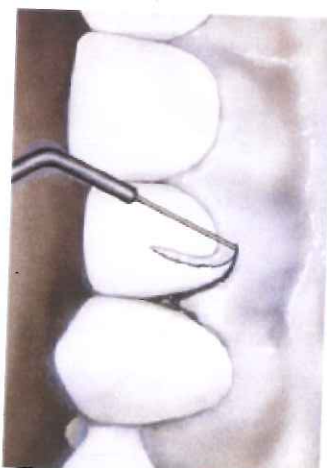Subgingival access and visibility

Using the Straight Knife electrode to restore gingival symmetry

Contour tissue with unparalleled precision

- ▶ With Perfect® TCS II, you can shave fine, "paper-thin" layers of tissue even in hard to reach areas
- ▶ Facilitate impression taking, cementation, and aesthetic procedures by removing redundant tissue with little or no bleeding

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“Coltène/Whaledent” PerFect TCS II Tissue Contouring System

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